



CE 0633



Mk4 Pro Personal Therapy System set up instructions.

Unpack both boxes carefully.

We suggest if possible that you retain the flat pack boxes in the unlikely event that the Therapy System has to be returned to Cyclo-ssage Ltd under warranty.

Lay the system on a flat surface.

The System can be placed on an existing bed, on a suitable massage table or on flat surface.

Assembly.

Using the set up diagram as an aid place in position the following numbered items.

1. The large top cushion using the zip.
2. The large leg elevation cushion under the leg area of the Therapy System
3. Small knobbed support cushion where required.
4. Infrared calf/foot cushion using the zip and Velcro.
5. Infrared lumbar cushion with Berlin-Cert logo-using the zip and Velcro.
6. Infrared shoulder/neck cushion-using the zip and Velcro.
7. The smaller upper body elevation cushion under the upper body area of the Therapy System.

Connecting the cables.

There are three push in cables protruding from the top, middle and bottom central area of the Therapy System.

Infrared - Shoulder / Neck Cushion cable.

Connect to the infrared shoulder / neck cushion to the top cable.
(Picture 6 on the diagram)

Infrared - Lumbar Cushion cable.

Connect to the infrared lumbar cushion with the Berlin-Cert logo to the middle cable.
(Picture 5 on the diagram)

Infrared - Calf / Foot Cushion cable.

Connect to the Calf / foot Infrared cushion to the bottom cable.
(Picture 4 on the diagram)

There are two cables protruding from the side of the Therapy System.

Digital hand control cable with a push in multi pin connector.

Connect the cable with a push in multi pin connector to the digital hand control.

12v power cable with a push in single connector.

This remaining (power) cable with a push in connector connects to the fixed lead that comes from the 12v transformer.

Connect the mains plug lead to the other end of the transformer.

Finally connect the mains plug into a suitable power supply and switch on.